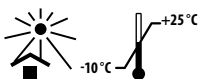
discleen® extra

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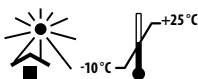
5 kg

EN discleen® extra

Instruction for use: For disinfection purposes, dilute in warm water (20–25 °C), to make a concentration of the solution shown in the table. A 1 % solution is prepared with dilution of 10 ml of concentrate in 990 ml of water. Used instruments and accessories are to be immersed fully in the prepared solution to ensure full coverage. After the prescribed time take the objects out of the solution, rinse with drinking water and give to sterilization. Do not use on medical devices made of silicone, latex, polycarbonate, softened PVC and natural synthetic rubber.

Active substances: N-(3-aminopropyl) N-dodecylpropane-1,3-diamine 66 g/kg (EC 219-145-8); Alkylbenzyltrimethylammonium chloride 35 g/kg (EC 270-325-2); Didecyltrimethylammonium chloride 50 g/kg (EC 230-525-2). **Other ingredients:** Guanidine, N,N'-1,3-propanediylbis-, N-coco alkylderivs., diacetates; alcohols C9-11 ethoxylated. **Attention: For professional use only. Danger! H302** Harmful if swallowed. **H314** Causes severe skin burns and eye damage. **H410** Very toxic to aquatic life with long lasting effect. **P273** Avoid release to the environment. **P280** Wear protective gloves / protective clothing / eye protection / face protection. **P301+P330+P331** IF SWALLOWED: rinse mouth. Do NOT induce vomiting. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a doctor. **First aid:** Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Inhalation - Shut off source of exposure. Bring the victim to the fresh air, keep at rest (avoid even walking), seek medical attention. **Storage:** Keep in original, well closed containers. Store in dry areas protected against weather conditions preventing possible release and entry of unauthorized persons. Do not keep exposed to direct sunlight. Store away from food, beverages and animal feedstuffs. Storage temperature: from -10 to +25 °C. **Disposal of the product and packaging:** Little amount of the product should be applied in harm with instructions for use. Larger quantities should be disposed of by competent company. Empty packagings. Contaminated packaging shall be handed over for disposal as hazardous waste. **Use before** date on the packaging.

Účinnost / Účinnost / Skuteczność / Efficacy	Norma / Norm	Konc. / Stężenie / Conc.		Čas / Czas / Time
Baktericidní / baktericidny / bakteriobójczy / bactericidal	EN 13727, EN 14561	0,5%	5 ml/l	5 min
Levurocidní / levurocidny / drożdżakobójczy / yeasticidal	EN 13624	0,25%	2,5 ml/l	15 min
	EN 14562	0,5%	5 ml/l	30 min
		1%	10 ml/l	15 min
Fungicidní / fungicidny / grzybobójczy / fungicidal (A.brasiliensis)	EN 13624, EN 14562	0,5%	5 ml/l	30 min
Virucidní MVA / virucidny MVA wirusobójczy wobec wirusa Vaccinia MVA / virucidal MVA	EN 14476, EN 17111	0,5%	5 ml/l	30 min
	EN 14476, EN 17111	1%	10 ml/l	15 min
Virucidní / virucidny / wirusobójczy / virucidal (Vaccinia)	DVV/RKI	0,75%	7,5 ml/l	60 min
		1%	10 ml/l	15 min
Virucidní / virucidny / wirusobójczy / virucidal (BVDV)	DVV/RKI	0,25%	2,5 ml/l	5 min
Mykobaktericidní / mykobaktericidny / prątkobójcze / mycobactericidal (M. terrae, M. avium)	EN 14348, prEN 14563	2%	20 ml/l	30 min

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